

Talking Tins™

A Problem Solved

Talking Tins is a new product designed to help the visually impaired identify the contents of their tinned food.

Identifying different objects of a similar shape has always been difficult for the visually impaired who want to stay independent. This frustrating problem is now solved by a new concept, using a re-recordable voice message to tell you what the item is.

This new product comes in the form of a magnetic cap that sits on top of any steel can. The cap allows you to record a short voice message, which can then be played back at any time with the touch of a button.





A wide range of applications

Multipurpose

The cap can also help you identify many other items. Our special accessories allow the cap to attach to other household objects such as Bottles, Sprays and Storage Containers. (See Accessories)

Features

- 1 Easy to use
- 2 Re-recordable & Reusable
- 3 High Quality voice reproduction
- 4 No barriers for language or regional accents
- 5 Built-in magnets secure the cap to any steel tin
- 6 Fits on all standard tin diameters
- 7 Fits on all tin tops, including Ring Pulls
- 8 Tins can be stacked one on top of each other
- 9 2 Record Buttons to provide message protection
- 10 Bright yellow housing and contrasting play button
- 11 Red Light to indicate recording
- 12 Large tactile Play Button
- 13 Replaceable button cell batteries
- 14 Use with Accessories to help identify other items

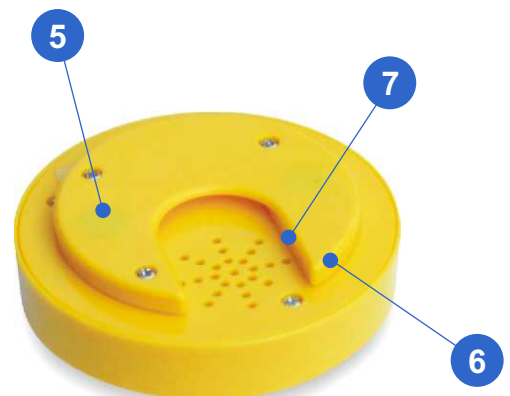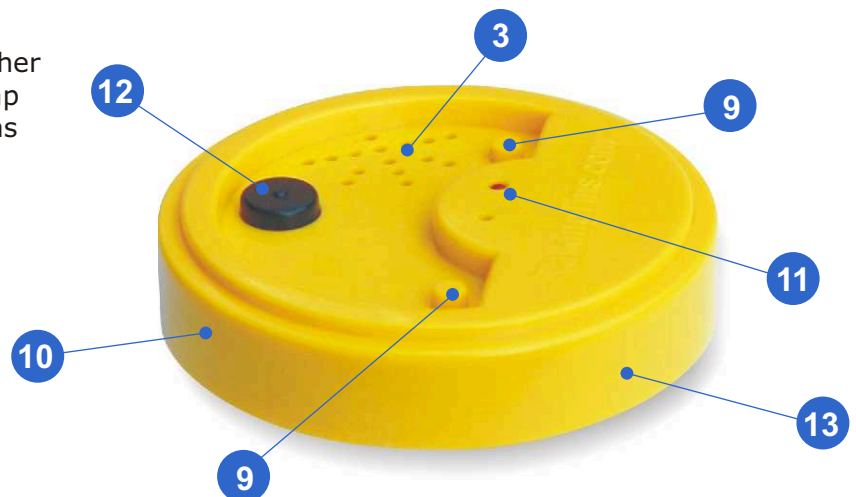

Helping You Stay Independent!

Easy to use

- Record a voice message into the cap to describe the contents of the tin and it's 'use by' date.
- Place the cap onto the tin and store as normal.
- Press the PLAY button at any time and the cap will playback your recorded message.



ACCESSORIES

The Talking Tins product can also help you identify many other items. Our special accessories allow the cap to attach to other household objects such as Bottles, Sprays and Storage Containers.

Elastic Strap

The Elastic Strap passes through the loop on the rear of the cap, allowing you to attach it to different shaped objects such as bottles and household sprays.



Metal Adapter

The Metal Adapter includes a self-adhesive pad, allowing you to attach it to any object. The cap will secure itself onto the adapter magnetically in the same way as it sits on top of a tin can.

Perfect for Storage Jars, Boxes and Containers



2 VERSIONS

The Talking Tin is available in either YELLOW with 10 seconds recording time or RED with 30 seconds recording



10 seconds recording




30 seconds recording

Helping You Stay Independent !

How to Use

Record a message in 3 easy steps

1. Hold the cap in both hands with your thumbs over the two record buttons. (See diagram below)
2. Press down with your thumbs and the red light will illuminate indicating that recording has begun. Record a short message to state the contents of the tin and it's 'use by' date.
3. Release your thumbs when you have finished recording. Press the PLAY button to hear your message. Your message can be up to 10 seconds long and can be re-recorded as many times as you wish.

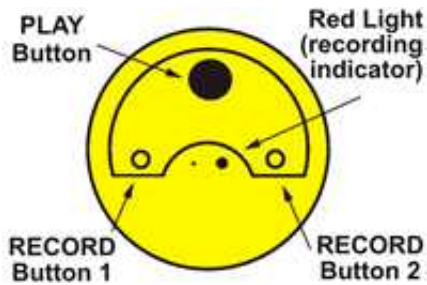

IMPORTANT: Ensure that both your thumbs remain pressed down so that the red light remains on whilst you are recording. Speak clearly when recording your message, at a distance of approximately 6 inches.

The cap fits magnetically on the top of any standard size steel tin can, including ring pull types.

If you require assistance, simply ask a friend or family member to help you record your messages.

Battery Replacement

You can purchase spare batteries from this web site or please ask your local distributor about their Battery Replacement Service. Batteries can be replaced by following the steps below:-

- a. Remove the U shaped section by loosening the 3 screws.
- b. Remove the screw holding the battery compartment cover.
- c. Replace the batteries and re-assemble in the reverse order.



Battery type:

AG13 Button Cell Set of 4 pieces

Please see our web site shop to purchase the batteries at discounted prices.

Talking Products Ltd

210 Mauretania Road, Nursling Ind. Estate,
SOUTHAMPTON, SO16 0YS, ENGLAND

Tel: +44 (0) 2380 730731

Fax: +44 (0) 2380 750227

Email: info@TalkingProducts.com

www.TalkingProducts.com

